


# The innovative applicator for instillation of surfactant



## NEOFACT®

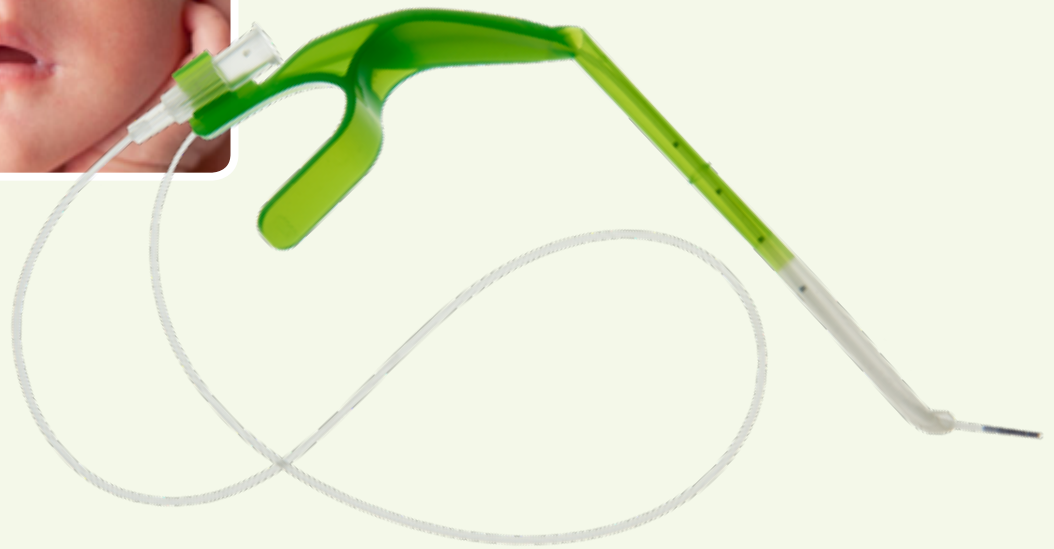

- Simple and practical handling
- Application of surfactant without Magill forceps
- Precise positioning in front of the glottis

# NEOFACT®

Makes the surfactant application easier for you –  
and less stressful for the child

## Using the Neofact® applicator:

1




The Neofact® applicator is supplied in sterile packaging for single use.

2



Use an intubation laryngoscope to check that the larynx is accessible.

3

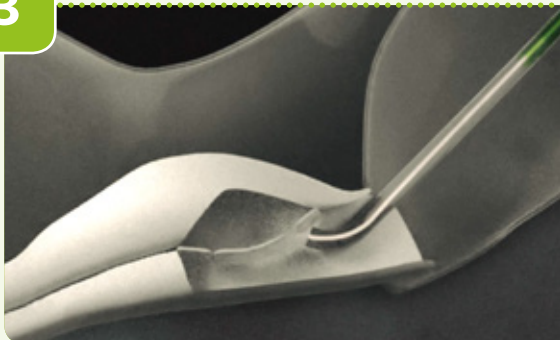

The Neofact® applicator can be operated only with one hand. Its angled tip enables precise positioning in front of the glottis – without the aid of Magill forceps.




4



Now simply push the catheter in carefully with your thumb.

5

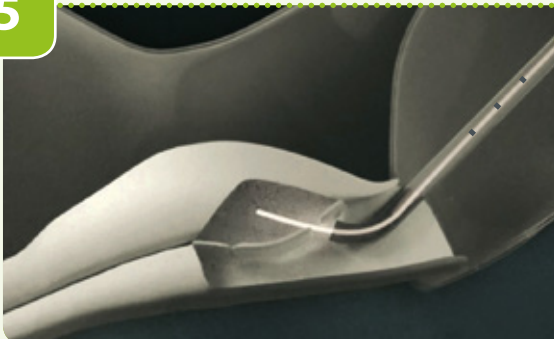

Use the scale on the applicator (1 cm increments) to precisely control the insertion depth.

6



Now instill Alveofact®: using the long catheter, Alveofact® can be injected beyond your field of view and operation. The Luer Lock system facilitates standardised connection.


Watch a video on the use of Neofact® here

[www.lyomark.com/neofact/](http://www.lyomark.com/neofact/)



# NEOFACT®

**Application device with tracheal catheter:  
for simple administration of surfactant –  
without Magill forceps**

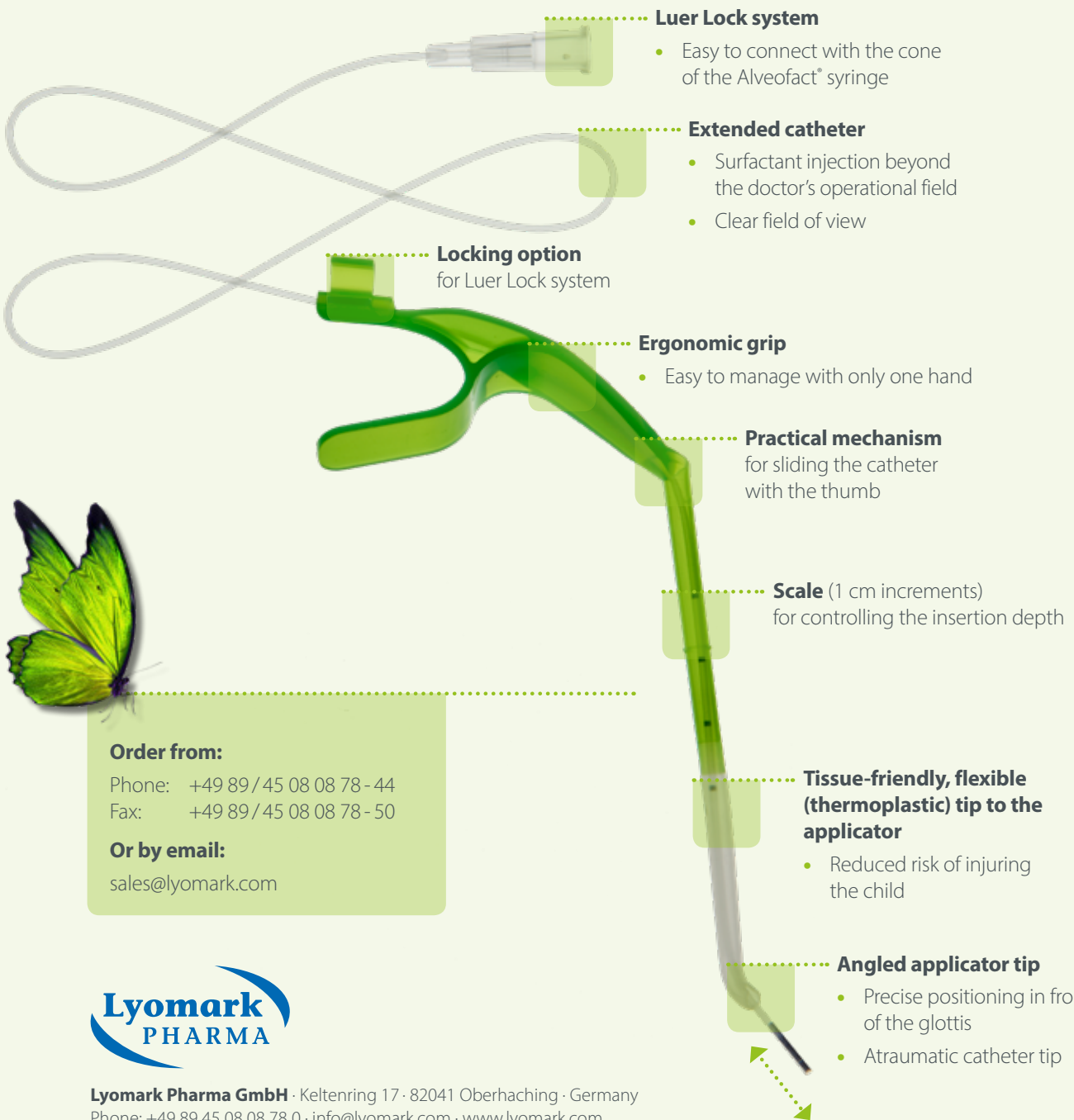

**Order from:**

Phone: +49 89 / 45 08 08 78 - 44

Fax: +49 89 / 45 08 08 78 - 50

**Or by email:**

[sales@lyomark.com](mailto:sales@lyomark.com)



**Lyomark Pharma GmbH** · Kelttenring 17 · 82041 Oberhaching · Germany  
Phone: +49 89 45 08 08 78 0 · [info@lyomark.com](mailto:info@lyomark.com) · [www.lyomark.com](http://www.lyomark.com)



**Murrplastik Medizintechnik GmbH** · Gewerbering 11 · 08223 Falkenstein · Germany

CE 0124